DESS LIBRARY INSTALLATION

FOR HYBRID ABUTMENTS, CROWNS, BRIDGES AND BARS

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
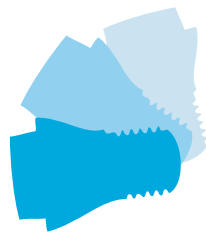
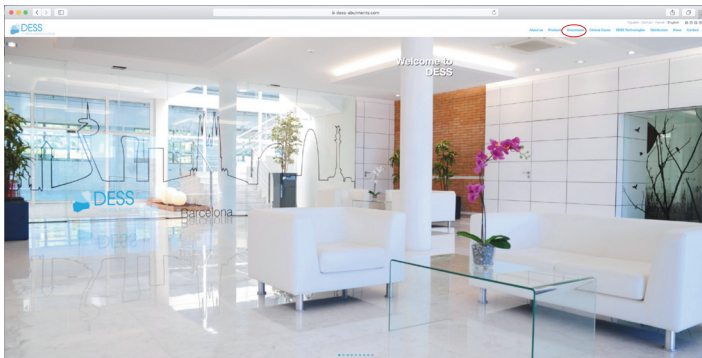
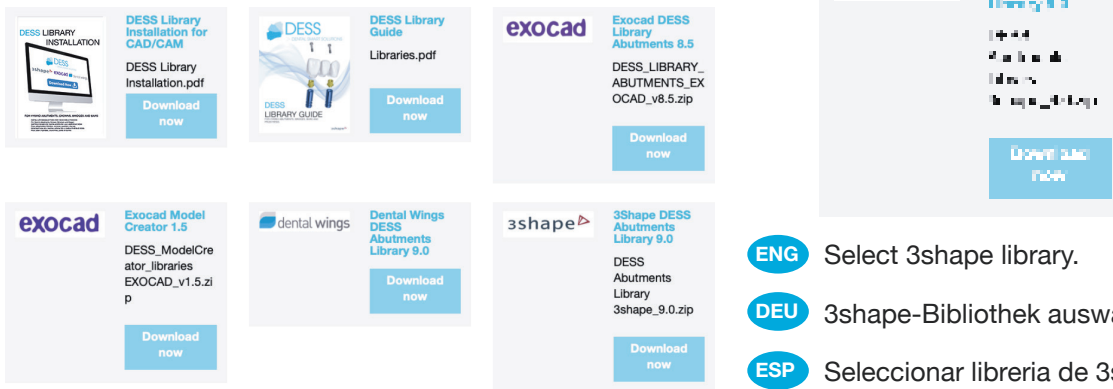
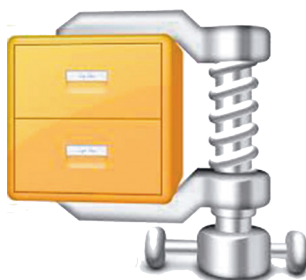
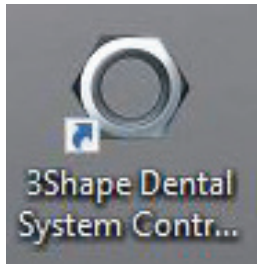
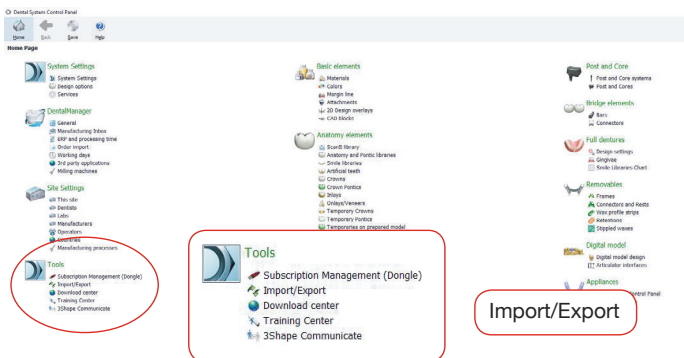
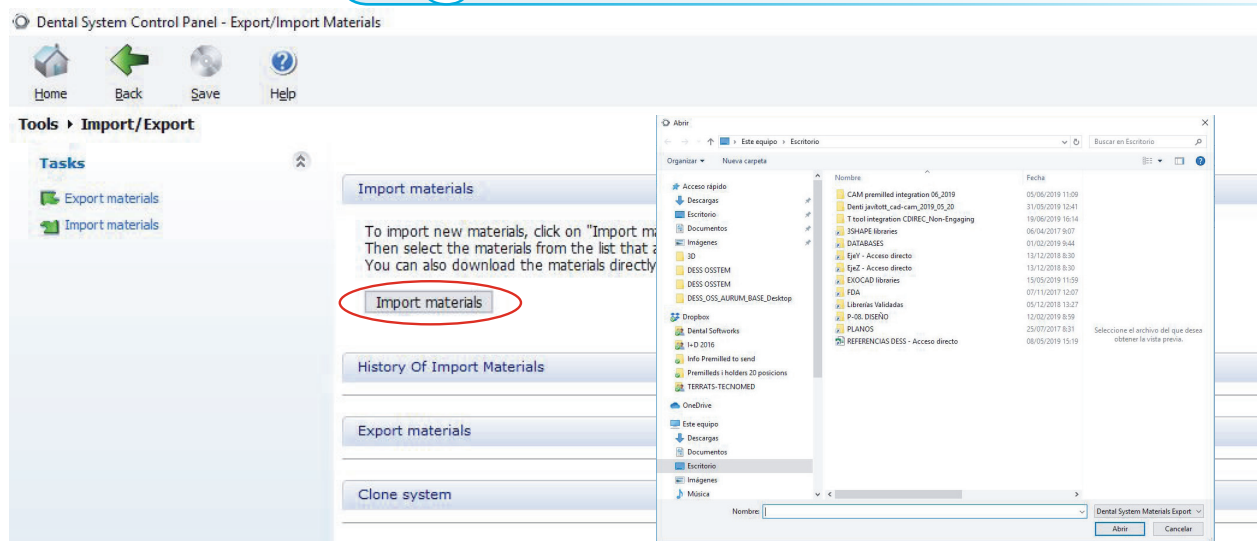
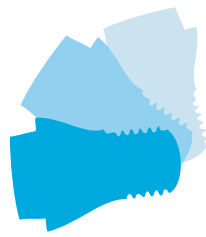
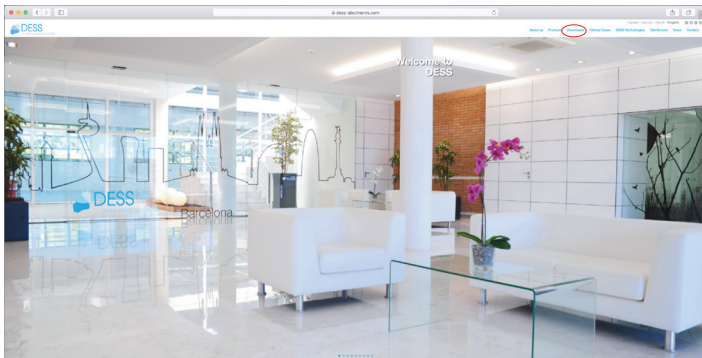
2

CAD CAM LIBRARIES

 DESS Library Installation for CAD/CAM DESS Library Installation.pdf Download now	 DESS Library Guide Libraries.pdf Download now	 Exocad DESS Library Abutments 8.5 DESS_LIBRARY_ABUTMENTS_EX OCAD_v8.5.zip Download now
 Exocad Model Creator 1.5 DESS_ModelCreator Libraries EXOCAD_v1.5.zip Download now	 Dental Wings DESS Abutments Library 9.0 Download now	 3Shape DESS Abutments Library 9.0 DESS Abutments Library 3shape_9.0.zip Download now

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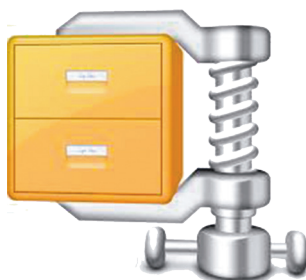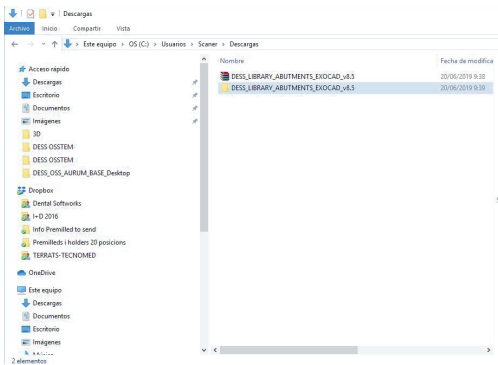
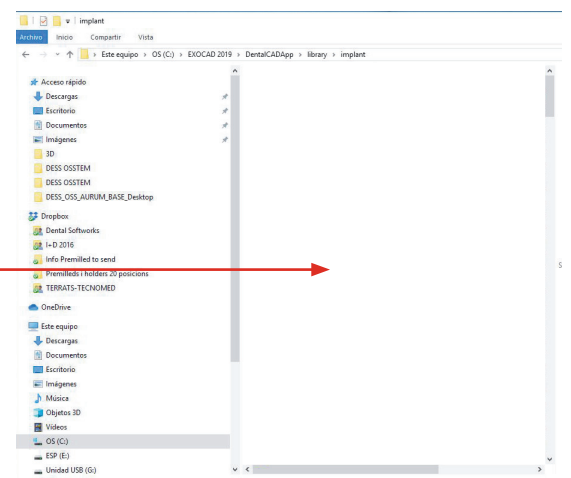
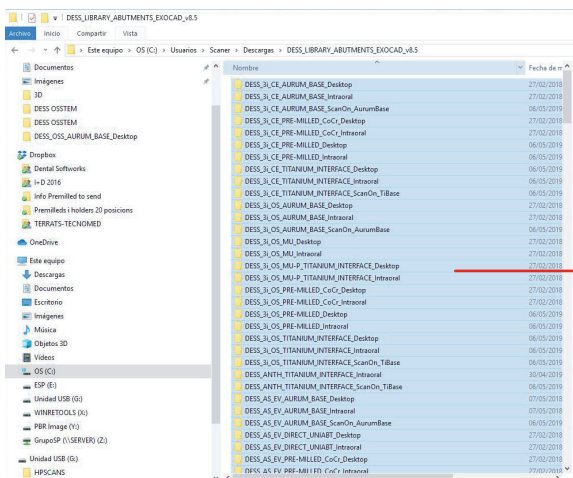
5


ENG Select all the folders and paste them in the following path: exocad / DentalCADApp/ library/implant.

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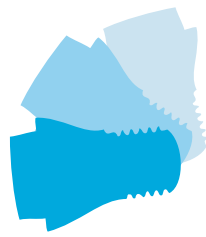
DESS

> DENTAL SMART SOLUTIONS

DESS MODEL CREATOR LIBRARIES **exocad**

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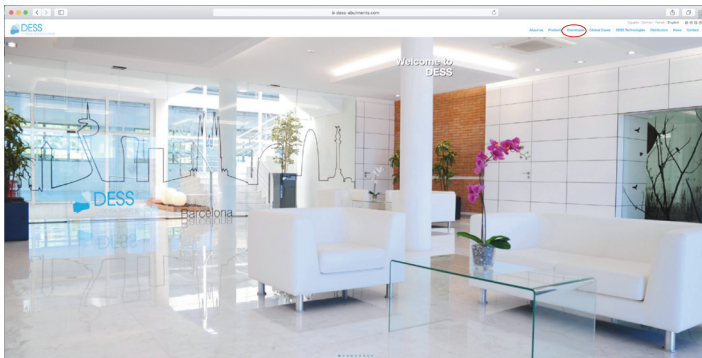
2

CAD CAM LIBRARIES

 DESS Library Installation for CAD/CAM DESS Library Installation.pdf Download now	 DESS Library Guide Libraries.pdf Download now	 Exocad DESS Library Abutments 8.5 DESS_LIBRARY_ABUTMENTS_EXOCAD_v8.5.zip Download now	 Exocad Model Creator 1.5 DESS_ModelCreator_libraries EXOCAD_v1.5.zip Download now
 Exocad Model Creator 1.5 DESS_ModelCreator_libraries EXOCAD_v1.5.zip Download now	 Dental Wings DESS Abutments Library 9.0 Download now	 3Shape DESS Abutments Library 9.0 DESS Abutments Library 3shape_9.0.zip Download now	


- ENG** Select exocad Model Creator.
- DEU** exocad Model Creator-Bibliothek auswählen.
- ESP** Seleccionar librería de exocad Model Creator.
- FR** Sélectionner la bibliothèque exocad Model Creator.

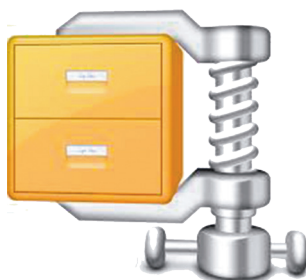3

- ENG** Unzip the folder.
- DEU** Dekomprimieren Sie die Datei.
- ESP** Descomprima el archivo.
- FR** Dézipper le dossier.

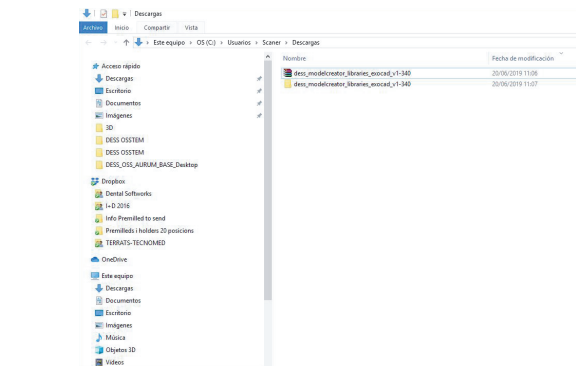4



ENG We searched in downloads the folder exocad and opened it.

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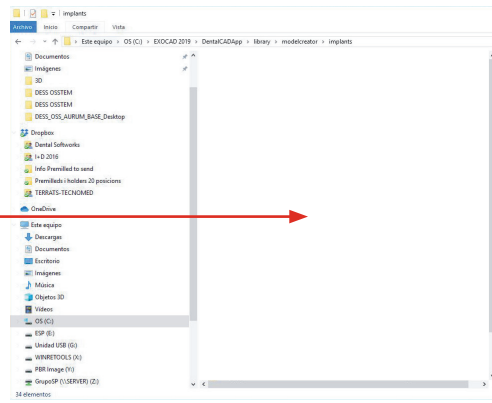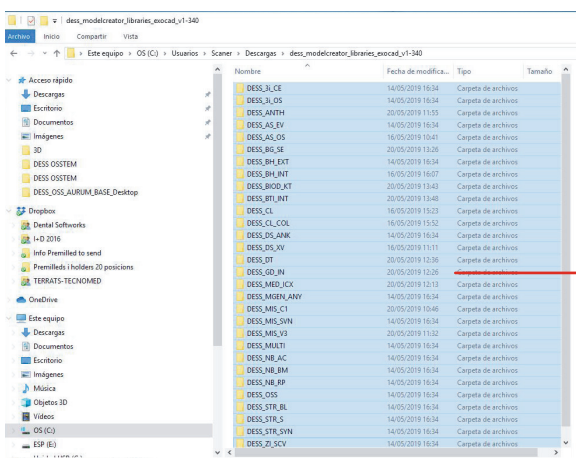5


ENG Select all the folders and paste them in the following path: exocad / DentalCADApp/ library/modelcreator/implant.

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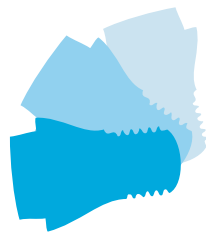
DESS

> DENTAL SMART SOLUTIONS

DESS LIBRARY INSTALLATION dental wings

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
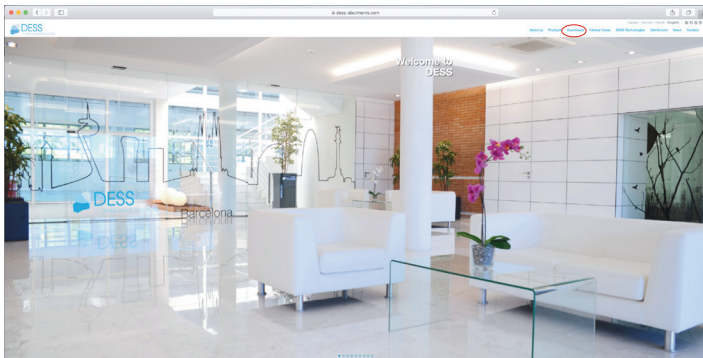
2

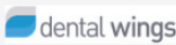CAD CAM LIBRARIES

 DESS Library Installation for CAD/CAM DESS Library Installation.pdf Download now	 DESS Library Guide Libraries.pdf Download now	 Exocad DESS Library Abutments 8.5 DESS_LIBRARY_ABUTMENTS_EXOCAD_v8.5.zip Download now	 Dental Wings DESS Abutments Library 9.0 Download now
 Exocad Model Creator 1.5 DESS_ModelCreator Libraries EXOCAD_v1.5.zip Download now	 Dental Wings DESS Abutments Library 9.0 Download now	 3Shape DESS Abutments Library 9.0 DESS Abutments Library 3shape_9.0.zip Download now	


- ENG** Select Dental wings library.
- DEU** Dental wings-Bibliothek auswählen.
- ESP** Seleccionar librería de Dental wings.
- FR** Sélectionner la bibliothèque Dental wings.

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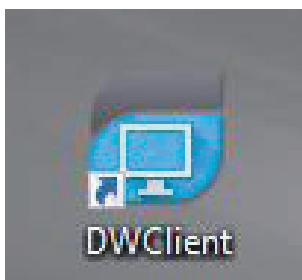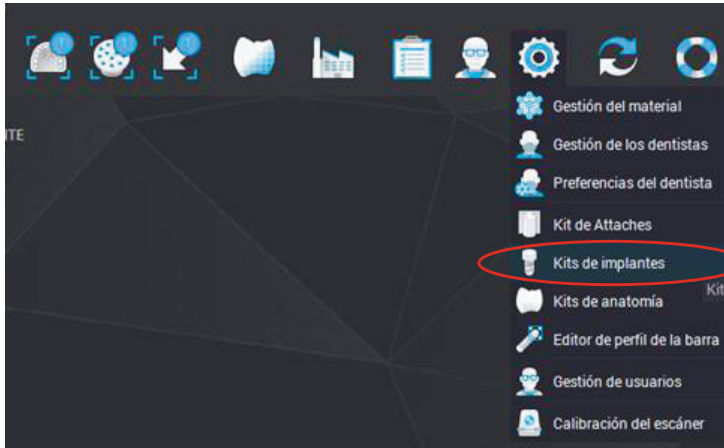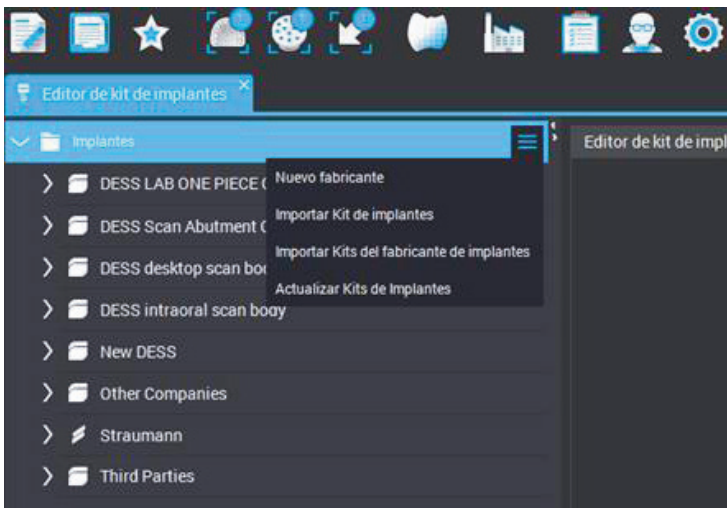
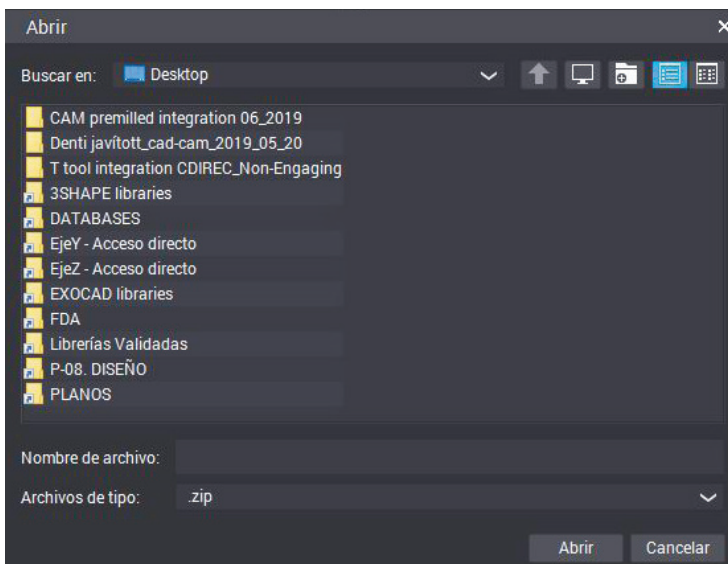
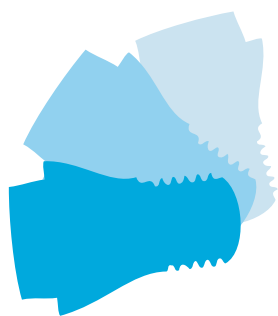
DESS

> DENTAL SMART SOLUTIONS

These instructions replace all previous editions. For any detailed information about DESS® products please contact your local distributor.

The photographs of the products are for information purposes only.

The type, reference and connection of each element should be checked. All DESS®

products are manufactured according to ISO 9001 and ISO 13485

with CE marking for all products Class IIa. | 3i® and 3i® Certain®, Miniplant®, MicroMiniplant™ and Osseotite®

are registered trademarks of 3i Implant Innovations, USA | ASTRA TECH™,

OsseoSpeed™, ANKYLOS® C/X, XiVE® and FRIADENT® and UniAbutment®

are registered trademarks of DENTSPLY Implants | BioHorizons® registered trademark of BioHorizons, INC.

| Brånemark System®, NobelReplace® and Replace Select™

and NobelActive™ and NobelReplace® CC and NobelSpeedy™

and Multi-unit® registered trademarks of Nobel Biocare AB, Sweden | ITI®, Straumann®

and synOcta® registered trademarks of Straumann Holding AG, Switzerland | MIS® registered trademark

of MIS Implant Technologies Ltd | Zimmer Screw-Vent®

and SwissPlus® registered trademarks of Zimmer Dental INC, USA | Dyna® registered trademark

of Dyna Dental Engineering BV | Locator® registered trademark

of Zest Anchors INC. | 3Shape® is a registered trademark of 3Shape A/S | EXOCAD® is a

registered trademark of Exocad GmbH, Germany | Dental Wings® and DWOS®

are registered trademarks of Dental Wings, Inc.