



DESS

> DENTAL SMART SOLUTIONS



DESS

LIBRARY GUIDE

FOR HYBRID ABUTMENTS, BRIDGES, BARS AND PROSTHESIS

exocad

Exocad library for implant level scan and abutment level scan **exocad**

For this CAD system the library draws distinction between the following groups:

- DESS (Connection name) Ti-base – for restorations connected with the implant via the Ti-base . The scanning needs to be performed with the Scan Body for the specific implant connection. **Note:** This group includes parts **scanned from abutment level** - Multi-Unit abutment, UniAbutment® and synOcta® abutment - with its specific Scan Abutment (marked almond in the table).
- DESS AS UNIABT - for restorations connected directly with the UniAbutment®. The scanning needs to be performed with the Scan Body for ASTRA TECH™ implant connection. The desired UniAbutment® will be mounted virtually in the CAD System. Please note that in this case the closing line between the prosthesis and the transepithelial abutment depends on the proper kit selection (abutment height in mm).
- DESS (Connection name) AURUMBase® – for restorations connected with the implant via the AURUM Base™ (see Table 1.3). The scanning needs to be performed with the Scan Body for the specific implant connection. **Note:** This group includes parts **scanned from abutment level** - Multi-Unit abutment - with its specific Scan Abutment (marked almond colour in the table).
- DESS Pre-milled Ti – for restorations connected with the implant via the pre-milled abutment made in Titanium (see Table 1.4). The scanning needs to be performed with the Scan Body for the specific implant connection. Note: The maximum width of the piece is restricted to 10 mm.
- DESS Pre-milled CoCr – for restorations connected with the implant via the pre-milled abutment made in CoCr (see Table 1.5). The scanning needs to be performed with the Scan Body for the specific implant connection. Note: The maximum width of the piece is restricted to 14 mm.

Take care to select the correct DESS Scan Abutment and the corresponding library file, as otherwise the milled prosthesis can differ from desired outcome and errors can occur, for example the level of interface settlement.

Table 1.1 DESS TI-Base

Name Library		Subtype	Implant Type	Analogue	Scan Abutments	
Nobel Active®	DESS - NB AC Ti-Base (Desktop or Intraoral or on Ti base)		3.0	14.068	52.068	
					53.100	
					52.068	
					52.200	
			NP/3.5	14.041	50.041	52.041
					53.100	
					50.041	52.041
					52.200	
			RP/4.3	14.042	50.042	52.042
					53.100	
					50.042	52.042
					52.200	
			WP/5.5	14.069	52.069	
					53.100	
					52.069	
					52.200	
Nobel Replace Select™	DESS - NB RP Ti-Base (Desktop or Intraoral or on Ti base)		NP/3.5	14.004	50.004	52.004
					53.100	
					50.004	52.004
					52.200	
			RP/4.3	14.005	50.005	52.005
					53.100	
					50.005	52.005
					52.200	
			WP/5.0	14.006	50.006	52.006
					53.100	
					50.006	52.006
					52.200	
			6.0	14.097	52.097	
					53.100	
					52.097	
					52.200	
Nobel Brånemark®	DESS - NB BM Ti-Base (Desktop or Intraoral or on Ti base)		NP/3.5	14.001	50.001	52.001
					53.100	
					50.001	52.001
					52.200	
			RP/4.1	14.002	50.002	52.002
					53.100	
					50.002	52.002
					52.200	
			WP/5.0	14.003	50.003	
					52.003	
					50.003	
					52.003	

 non-engaging

 non-engaging

 Scan Abutment
for desktop scanning
50.0XX

 Scan Abutment
for intra-oral scanning
52.0XX

 Scan Abutment
on Ti base
53.100 53.200

Name Library		Subtype	Implant Type	Analogue	Scan Abutments	
3i Osseotite®	DESS - 3i OS Ti-Base (Desktop or Intraoral or on Ti base)		NP/3.4	14.011	50.011	52.011
					53.100	
					50.011	52.011
					52.200	
			RP/4.1	14.012	50.012	52.012
					53.100	
					50.012	52.012
					52.200	
			WP/5.0	14.013	50.013	52.013
					53.100	
3i Certain®	DESS - 3i CE Ti-Base (Desktop or Intraoral or on Ti base)		NP/3.4	14.014	50.014	52.014
					53.100	
					50.014	52.014
					52.200	
			RP/4.1	14.015	50.015	52.015
					53.100	
					50.015	52.015
					52.200	
			WP/5.0	14.016	50.016	52.016
					53.100	
Straumann® Soft Tissue Level	DESS - STR S Ti-Base (Desktop or Intraoral or on Ti base)		NNC/3.5	14.070	52.069	
					53.100	
					52.069	
					52.200	
			RN/4.8	14.009	50.009	
					52.009	
					50.009	
					52.009	
			RN on synOCTA®	14.010	50.010	
					52.010	
					50.010	
					52.010	
			WN/6.5	14.045	50.045	
					52.045	
					50.045	
					52.045	
			WN on synOCTA®	14.046	50.015	
					52.015	
					50.015	
					52.015	

 non-engaging

 non-engaging

 Scan Abutment
for desktop scanning
50.OXX

 Scan Abutment
for intra-oral scanning
52.OXX

 Scan Abutment
on Ti base
53.100 53.200

Name Library		Subtype	Implant Type	Analogue	Scan Abutments	
Straumann® Bone Level	DESS - STR BL Ti-Base (Desktop or Intraoral or on Ti base)		NC/3.3	14.043	50.043	52.043
					53.100	
			RC/4.1	14.044	50.043	52.043
					52.200	
			SRA	14.098	50.044	52.044
					52.200	
Zimmer Screw-Vent®	DESS - ZI SCV Ti-Base (Desktop or Intraoral or on Ti base)		NP/3.5	14.017	52.098	
					52.098	
			RP/4.5	14.018	50.017	52.017
					53.100	
			WP/5.7	14.019	50.017	52.017
					52.200	
Astra tech implant system™ EV	DESS - AS EV Ti-Base (Desktop or Intraoral or on Ti base)		EV/3.6	14.059	50.018	52.018
					53.100	
			EV/4.2	14.060	50.018	52.018
					52.200	
			EV/4.8	14.061	50.019	52.019
					53.100	
			Uniabutment®	14.058	50.019	52.019
					52.200	

 non-engaging

 non-engaging

 Scan Abutment
for desktop scanning
50.0XX

 Scan Abutment
for intra-oral scanning
52.0XX

 Scan Abutment
on Ti base
53.100 53.200

Name Library		Subtype	Implant Type	Analogue	Scan Abutments	
Astra tech osseospeed™	DESS - AS Ti-Base (Desktop or Intraoral or on Ti base)	 	NP/3.0	14.023	52.023	
					53.100	
					50.059	52.059
		 	RP/3.5-4.0	14.024	52.200	
					50.024	52.024
					53.100	
		 	WP/4.5-5.0	14.025	50.024	52.024
					52.200	
					50.025	52.025
		 	Uniabutment® 20°	14.036	50.025	52.025
					53.100	
		 	Uniabutment® 45°	14.037	50.036	52.036
					52.037	
Friadent XIVE®	DESS - DS XV Ti-Base (Desktop or Intraoral or on Ti base)	 	NP/3.4	14.038	50.038	52.038
					53.100	
					50.038	52.038
		 	RP/3.8	14.039	52.200	
					50.039	52.039
					53.100	
		 	WP/4.5	14.040	50.039	52.039
					52.200	
					50.040	52.040
		 	5.5	14.071	50.040	52.040
					53.100	
					52.071	
Ankylos® C/X	DESS - DS ANK Ti-Base (Desktop or Intraoral or on Ti base)	 	All platforms, height GH 1,5	14.048 14.049	52.071	52.200
					50.049	52.049
					53.100	
		 	All platforms, height GH 2,5	14.048 14.049	50.049	52.049
					53.100	
					52.200	

 non-engaging

 non-engaging

 Scan Abutment
for desktop scanning
50.0XX

 Scan Abutment
for intra-oral scanning
52.0XX

 Scan Abutment
on Ti base
53.100 53.200

Name Library		Subtype	Implant Type	Analogue	Scan Abutments	
Osstem® TS	DESS - OSS Ti-Base (Desktop or Intraoral or on Ti base)		Mini Yellow	14.072	50.072	52.072
					53.100	
			STD Regular	14.073	50.072	52.072
					52.200	
					50.073	52.073
					53.100	
Camlog®	DESS - CL Ti-Base (Desktop or Intraoral or on Ti base)		3.3	14.064	50.073	52.073
					52.200	
			3.8	14.065	50.064	52.064
					52.200	
					50.065	52.065
			4.3	14.066	53.100	
					50.065	52.065
					52.200	
			5.0	14.067	50.066	52.066
					53.100	
					50.066	52.066
					52.200	
Conelog®	DESS - CL COL Ti-Base (Desktop or Intraoral or on Ti base)		NP/3.3	14.074	52.067	52.067
					53.100	
			RP/3.8	14.075	52.074	52.074
					52.200	
					52.075	52.075
			WP/5.0	14.076	52.200	
					52.076	52.076
					53.100	
					52.076	52.076

 non-engaging

 non-engaging

 Scan Abutment
for desktop scanning
50.0XX

 Scan Abutment
for intra-oral scanning
52.0XX

 Scan Abutment
on Ti base
53.100 53.200

Name Library		Subtype	Implant Type	Analogue	Scan Abutments	
Biohorizns® Internal	DESS - BH INT Ti-Base (Desktop or Intraoral or on Ti base)	 	NP/3.5	14.087	52.087	53.100
					52.087	52.200
		 	RP/4.5	14.088	52.088	53.100
					52.088	52.200
		 	WP/5.7	14.089	52.089	53.100
					52.089	52.200
	DESS - BH EXT Ti-Base (Desktop or Intraoral or on Ti base)	 	NP/3.7	14.020	50.020	52.020
					53.100	52.020
		 	RP/4.2	14.021	50.021	52.021
					53.100	52.021
		 	WP/5.2	14.022	50.022	52.022
					50.022	52.022
MIS® Seven	DESS - MIS SVN Ti-Base (Desktop or Intraoral or on Ti base)	 	NP/3.7	14.056	50.056	52.056
					53.100	52.056
		 	SP/4.2	14.095	50.017	52.017
					53.100	52.017
		 	WP/5.2	14.096	50.018	52.018
					53.100	52.018
		 			50.018	52.018
					52.200	

 non-engaging

 non-engaging

 Scan Abutment
for desktop scanning
50.0XX

 Scan Abutment
for intra-oral scanning
52.0XX

 Scan Abutment
on Ti base
53.100 53.200

Name Library		Subtype	Implant Type	Analogue	Scan Abutments				
MIS® C1 internal	DESS - MIS C1 Ti-Base (Desktop or Intraoral or on Ti base)		NP	14.081	52.081				
					53.100				
			SP	14.082	52.081				
					52.200				
			WP	14.083	52.082				
					53.100				
			Multiunit	14.085	52.082				
					52.200				
		MIS® V3	DESS - MIS V3 Ti-Base (Desktop or Intraoral or on Ti base)		NP	14.084	52.083		
							53.100		
	SP			14.082	52.084				
					52.200				
	Multiunit			14.085	52.083				
					53.100				
Megagen AntRidge®	DESS - MG ANY Ti-Base (Desktop or Intraoral or on Ti base)					14.057	52.082		
			52.200						
		Anthogyr AXIOM®	DESS - ANTH Ti-Base (Desktop or Intraoral or on Ti base)					14.077	50.057
									53.100
Medentis ICX®	DESS - MED ICX Ti-Base (Desktop or Intraoral or on Ti base)					14.080			50.077
									52.200
									50.080
									53.100
50.080									
52.200									
non-engaging non-engaging Scan Abutment for desktop scanning 50.0XX Scan Abutment for intra-oral scanning 52.0XX Scan Abutment on Ti base 53.100 53.200									

	<i>Name Library</i>	<i>Subtype</i>	<i>Platform</i>	<i>Analogue</i>	<i>Scan Abutments</i>
GlobalID IN-KONE®	DESS - GD INT Ti-Base (Desktop or Intraoral or on Ti base)			14.086	52.086
					53.100
					52.086
					52.200
Dentium Superline™ & Implantium	DESS - DT Ti-Base (Desktop or Intraoral)			14.090	52.090
					53.100
					52.090
					52.200
Bego Semados®	DESS - BG SE Ti-Base (Desktop or Intraoral or on Ti base)		3.25/3.75	14.091	52.091
					53.100
					52.091
					52.200
			4.1	14.092	52.092
					53.100
					52.092
					52.200
			4.5	14.093	52.093
					53.100
					52.093
					52.200
			5.5	14.094	52.094
					53.100
					52.094
					52.200
Biotech Kontakt®	DESS - BIOD KT Ti-Base (Desktop or Intraoral or on Ti base)			14.078	52.078
					53.100
					52.078
					52.200
BTI®	DESS - BTI INT Ti-Base (Desktop or Intraoral or on Ti base)			14.079	52.079
					53.100
					52.079
					52.200
Multi-Unit	DESS - Multi Ti-Base (Desktop or Intraoral or on Ti base)		RP 15.007	14.007	50.007
					52.007
			WP 15.054	14.047	50.047
					52.047



non-engaging



non-engaging



Scan Abutment
for desktop scanning
50.0XX



Scan Abutment
for intra-oral scanning
52.0XX



Scan Abutment
on Ti base
53.100 53.200

**Table 1.2 DESS
scan on Uniabutment &
Multi-Unit & Synocta**

	<i>Category</i>	<i>Type</i>	<i>Platform</i>	<i>Analogue</i>	<i>Scan Abutments</i>
Uniabutment® on Astra Tech™ Osseospeed™	DESS UNIABUTMENT DIRECT (Uniabutment Scan Abutment)		Uniabutment® 20°	14.036	50.036 52.036
	DESS UNIABUTMENT DIRECT (Uniabutment Scan Abutment)		Uniabutment® 45°	14.037	50.037 52.037
Astra Tech™ Implant System™ EV	DESS UNIABUTMENT DIRECT (Uniabutment Scan Abutment)		Uniabutment®	14.058	50.058 52.058

Multi-Unit	DESS MULTIUNIT DIRECT (Multi-Unit Scan Abutment)		RP	14.007	50.036 52.036
	DESS MULTIUNIT DIRECT (Multi-Unit Scan Abutment)		WP	14.047	50.037 52.037

Straumann® Soft Tissue level witch synOcta® Abutment	DESS STR SYN DIRECT (Bars and bridges)		RN on synOcta®	14.010	50.010 52.010
	DESS STR SYN DIRECT (Bars and bridges)		WN on synOcta®	14.046	50.046 52.037

 non-engaging

 non-engaging

 Scan Abutment
for desktop scanning
50.OXX

 Scan Abutment
for intra-oral scanning
52.OXX

Table 1.3 DESS ANGLEBase®

	Name Library	Subtype	Platform	Analogue	Scan Abutments	
Nobel Active®	DESS - NB AC AURUM BASE (Desktop or Intraoral or on Ti base)		NP	14.041	50.041	52.041
					53.100	
					50.041	52.041
			RP	14.042	52.200	
					50.042	52.042
					53.100	
Nobel Replace Select™	DESS - NB RP AURUM BASE (Desktop or Intraoral or on Ti base)		NP	14.004	50.042	52.042
					53.100	
					50.042	52.042
			RP	14.005	52.200	
					50.042	52.042
					53.100	
			WP	14.006	50.042	52.042
					53.100	
					50.042	52.042
Nobel Brånemark®	DESS - NB BM AURUM BASE (Desktop or Intraoral or on Ti base)		NP	14.001	50.004	52.004
					53.100	
					50.004	52.004
			RP	14.002	52.200	
					50.005	52.005
					53.100	
			WP	14.006	50.005	52.005
					52.200	
					50.005	52.005
3i Osseotite®	DESS - 3i OS AURUM BASE (Desktop or Intraoral or on Ti base)		NP/3.4	14.004	50.006	52.006
					53.100	
					50.006	52.006
			RP/4.1	14.005	52.200	
					50.006	52.006
					53.100	
			WP/5.0	14.006	50.006	52.006
					52.200	
					50.006	52.006

 non-engaging

 non-engaging

 Scan Abutment
for desktop scanning
50.0XX

 Scan Abutment
for intra-oral scanning
52.0XX

 Scan Abutment
on Ti base
53.100 53.200

	Name Library	Subtype	Platform	Analogue	Scan Abutments	
3i Certain®	DESS - 3i CE AURUM BASE (Desktop or Intraoral or on Ti base)		NP/3.4	14.014	50.014	52.014
					53.100	
			RP/4.1	14.015	50.014	52.014
					52.200	
			WP/5.0	14.016	50.015	52.015
					53.100	
Straumann® Soft Tissue Level	DESS - STR S AURUM BASE (Desktop or Intraoral or on Ti base)		RN/4.8	14.009	50.016	52.016
					53.100	
			WN/6.5	14.045	50.015	52.015
					52.200	
					50.016	52.016
					52.200	
Straumann® Bone Level	DESS- STR BL AURUM BASE (Desktop or Intraoral or on Ti base)		NC/3.3	14.043	50.009	52.009
					53.100	
			RC/4.1	14.044	50.009	52.009
					53.100	
					50.045	52.045
					52.045	
Zimmer Screw-Vent®	DESS - ZI SCV AURUM BASE (Desktop or Intraoral or on Ti base)		NP/3.5	14.017	50.043	52.043
					53.100	
			RP/4.5	14.018	50.043	52.043
					52.200	
			WP/5.7	14.019	50.044	52.044
					53.100	

 non-engaging

 non-engaging

 Scan Abutment
for desktop scanning
50.0XX

 Scan Abutment
for intra-oral scanning
52.0XX

 Scan Abutment
on Ti base
53.100 53.200

	Name Library	Type	Platform	Analogue	Scan Abutments	
Astra tech implant system™ EV	DESS - AS EV Ti-Base AURUM BASE (Desktop or Intraoral or on Ti base)	 	EV/3.6	14.059	50.059	52.059
					53.100	
		 	EV/4.2	14.060	50.059	52.059
					52.200	
		 	EV/4.8	14.061	50.060	52.060
					53.100	
		 			50.060	52.060
					52.200	
		 			50.061	52.061
					53.100	
Astra tech osseospeed™	DESS - AS AURUM BASE (Desktop or Intraoral or on Ti base)	 	RP	14.024	50.024	52.024
					53.100	
		 	WP	14.025	50.024	52.024
					52.200	
		 			50.025	52.025
					53.100	
Friadent XIVE®	DESS - DS XV AURUM BASE (Desktop or Intraoral or on Ti base)	 	NP	14.038	50.025	52.025
					52.200	
		 	RP	14.039	50.038	52.038
					53.100	
		 	WP	14.040	50.039	52.039
					52.200	
		 			50.039	52.039
					52.200	
		 			50.040	52.040
					53.100	

 non-engaging

 non-engaging

 Scan Abutment
for desktop scanning
50.0XX

 Scan Abutment
for intra-oral scanning
52.0XX

 Scan Abutment
on Ti base
53.100 53.200

	Name Library	Type	Platform	Analogue	Scan Abutments	
Osstem® TS	DESS - OSS STD AURUM BASE (Desktop or Intraoral or on Ti base)		MINI	14.072	50.072	52.072
					53.100	
					50.072	52.072
			STD	14.073	53.200	
					50.073	52.073
					53.100	
Camlog®	DESS - CL AURUM BASE (Desktop or Intraoral or on Ti base)		3.8	14.065	50.073	52.073
					53.100	
					50.073	52.073
			4.3	14.066	53.200	
					50.073	52.073
					53.100	
Mis® Seven	DESS - MIS SVN AURUM BASE (Desktop or Intraoral or on Ti base)		SP	14.095	50.065	52.065
					53.100	
					50.065	52.065
			WP	14.066	53.200	
					50.066	52.066
					53.100	
Megagen AnyRidge®	DESS - MG ANY AURUM BASE (Desktop or Intraoral or on Ti base)			14.096	50.017	52.017
					53.100	
					50.017	52.017
Medentis ICX®	DESS - MED ICX AURUM BASE (Desktop or Intraoral or on Ti base)			14.057	53.200	
					50.018	52.018
					53.100	
Multi-Unit	DESS - Multi AURUM BASE (Desktop or Intraoral or on Ti base)		RP	14.096	50.018	52.018
			WP	14.080	53.200	
					50.018	52.018



non-engaging



non-engaging



Scan Abutment
for desktop scanning
50.0XX



Scan Abutment
for intra-oral scanning
52.0XX



Scan Abutment
on Ti base
53.100 53.200

Table 1.4 DESS Pre-milled Abutment Ti

Name Library		Implant Type	Analogue	Scan Abutments	
Nobel Active®	DESS - NB AC Pre-milled Ti (Desktop or Intraoral)	3.0	14.068	52.068	53.100
		NP	14.041	50.041	52.041
		RP	14.042	50.042	52.042
		WP	14.069	52.069	53.100
				53.100	
Nobel Replace Select™	DESS - NB RP Pre-milled Ti (Desktop or Intraoral)	NP	14.004	50.004	52.004
		RP	14.005	50.005	52.005
		WP	14.006	50.006	52.006
		6.0	14.097	52.097	53.100
				53.100	
Nobel Brånemark®	DESS - NB BM Pre-milled Ti (Desktop or Intraoral)	NP	14.001	50.001	52.001
		RP	14.002	50.002	52.002
		WP	14.003	50.003	52.003
3i Osseotite®	DESS - 3i OS Pre-milled Ti (Desktop or Intraoral)	NP	14.011	50.011	52.011
		RP	14.012	50.012	52.012
		WP	14.013	50.013	52.013
3i Certain®	DESS - 3i CE Pre-milled Ti (Desktop or Intraoral)	NP	14.014	50.014	52.014
		RP	14.015	50.015	52.015
		WP	14.016	50.016	52.016
Straumann® Soft Tissue Level	DESS - STR S Pre-milled Ti (Desktop or Intraoral)	NNC	14.070	52.069	53.100
		RN	14.009	50.009	52.009
		WN	14.045	50.045	52.045

 non-engaging

 non-engaging

 Scan Abutment for desktop scanning
50.0XX

 Scan Abutment for intra-oral scanning
52.0XX

 Scan Abutment on Ti base
53.100 53.200

Name Library		Implant Type	Analogue	Scan Abutments	
Straumann® Bone Level	DESS - STR BL Pre-milled Ti (Desktop or Intraoral)	NC	14.043	50.043 53.100	52.043
		RC	14.044	50.044 53.100	52.044
Zimmer Screw-Vent®	DESS - ZI SCV Pre-milled Ti (Desktop or Intraoral)	NP	14.017	50.017 53.100	52.017
		RP	14.018	50.018 53.100	52.018
		WP	14.019	50.019 53.100	52.019
Astra tech implant system™ EV	DESS - AS EV Pre-milled Ti (Desktop or Intraoral)	EV/3.6	14.059	50.059 53.100	52.059
		EV/4.2	14.060	50.060 53.100	52.060
		EV/4.8	14.061	50.061 53.100	52.061
Astra tech implant Osseospeed™	DESS - AS OS Pre-milled Ti (Desktop or Intraoral)	RP	14.024	50.024 53.100	52.024
		WP	14.025	50.025 53.100	52.025
Friadent XIVE®	DESS - XV Pre-milled Ti (Desktop or Intraoral)	NP	14.038	50.038 53.100	52.038
		RP	14.039	50.039 53.100	52.039
		WP	14.040	50.040 53.100	52.040
Ankylos® C/X	DESS - ANK Pre-milled Ti (Desktop or Intraoral)	All platforms, height GH 2,5	14.048 14.049	50.049 53.100	52.049
Osstem® TS	DESS - OS Pre-milled Ti (Desktop or Intraoral)	Mini Yellow	14.072	50.072 53.100	52.072
		STD Regular	14.073	50.073 53.100	52.073

 non-engaging

 non-engaging

 Scan Abutment
for desktop scanning
50.0XX
 Scan Abutment
for intra-oral scanning
52.0XX
 Scan Abutment
on Ti base
53.100 53.200

Name Library		Implant Type	Analogue	Scan Abutments	
Camlog®	DESS - CL Pre-milled Ti (Desktop or Intraoral)	3.3	14.064	52.064	53.100
		3.8	14.065	50.065	52.065
		4.3	14.066	50.066	52.066
		5.0	14.067	52.067	53.100
Conelog®	DESS - CL COL Pre-milled Ti (Desktop or Intraoral)	NP	14.074	52.074	53.100
		RP	14.075	52.075	53.100
		WP	14.076	52.076	53.100
Biohorizns® External	DESS - BH EXT Pre-milled Ti (Desktop or Intraoral)	RP/4.2	14.021	50.021	52.021
		WP/5.2	14.022	50.022	52.022
MIS® Seven	DESS - MIS SVN Pre-milled Ti (Desktop or Intraoral)	SP	14.095	50.017	52.017
		WP	14.096	50.018	52.018
Megagen AntRidge®	DESS - MG Pre-milled Ti (Desktop or Intraoral)		14.057	50.057	53.100
Dentium Superline™ & Implantium	DESS - DT Pre-milled Ti (Desktop or Intraoral)	RP	14.090	52.090	53.100

 non-engaging

 non-engaging

 Scan Abutment
for desktop scanning
50.0XX

 Scan Abutment
for intra-oral scanning
52.0XX

 Scan Abutment
on Ti base
53.100 53.200

**Table 1.5 DESS
Pre-milled Abutment
CoCr**

Name Library		Implant Type	Analogue	Scan Abutments	
Nobel Active®	DESS - NB AC Pre-milled CoCr (Desktop or Intraoral)	3.0	14.068	52.068	53.100
		NP	14.041	50.041	52.041
		RP	14.042	50.042	52.042
		WP	14.069	52.069	53.100
				53.100	
Nobel Replace Select™	DESS - NB RP Pre-milled CoCr (Desktop or Intraoral)	NP	14.004	50.004	52.004
		RP	14.005	50.005	52.005
		WP	14.006	50.006	52.006
		6.0	14.097	52.097	53.100
				53.100	
Nobel Brånemark®	DESS - NB BM Pre-milled CoCr (Desktop or Intraoral)	NP	14.001	50.001	52.001
		RP	14.002	50.002	52.002
		WP	14.003	50.003	52.003
3i Osseotite®	DESS - 3i OS Pre-milled CoCr (Desktop or Intraoral)	NP	14.011	50.011	52.011
		RP	14.012	50.012	52.012
		WP	14.013	50.013	52.013
3i Certain®	DESS - 3i CE Pre-milled CoCr (Desktop or Intraoral)	NP	14.014	50.014	52.014
		RP	14.015	50.015	52.015
		WP	14.016	50.016	52.016
Straumann® Soft Tissue Level	DESS - STR S Pre-milled CoCr (Desktop or Intraoral)	RN	14.009	50.009	52.009
		WN	14.045	50.045	52.045

 non-engaging

 non-engaging

 Scan Abutment for desktop scanning
50.0XX

 Scan Abutment for intra-oral scanning
52.0XX

 Scan Abutment on Ti base
53.100 53.200

	<i>Name Library</i>	<i>Implant Type</i>	<i>Analogue</i>	<i>Scan Abutments</i>	
Straumann® Bone Level	DESS - STR BL Pre-milled CoCr (Desktop or Intraoral)	NC	14.043	50.043	52.043
		RC	14.044	50.044	52.044
Zimmer Screw-Vent®	DESS - ZI SCV Pre-milled CoCr (Desktop or Intraoral)	NP	14.017	50.017	52.017
		RP	14.018	50.018	52.018
		WP	14.019	50.019	52.019
Astra tech implant system™ EV	DESS - AS EV Pre-milled CoCr (Desktop or Intraoral)	EV/3.6	14.059	50.059	52.059
		EV/4.2	14.060	50.060	52.060
		EV/4.8	14.061	50.061	52.061
Astra tech implant Osseospeed™	DESS - AS OS Pre-milled CoCr (Desktop or Intraoral)	RP	14.024	50.024	52.024
		WP	14.025	50.025	52.025
Friadent XIVE®	DESS - XV Pre-milled CoCr (Desktop or Intraoral)	NP	14.038	50.038	52.038
		RP	14.039	50.039	52.039
		WP	14.040	50.040	52.040
Ankylos® C/X	DESS - ANK Pre-milled CoCr (Desktop or Intraoral)	All platforms, height GH 2,5	14.048 14.049	50.049	52.049
Osstem® TS	DESS - OS Pre-milled CoCr (Desktop or Intraoral)	Mini Yellow	14.072	50.072	52.072
		STD Regular	14.073	50.073	52.073

 non-engaging

 non-engaging

 Scan Abutment
for desktop scanning
50.0XX

 Scan Abutment
for intra-oral scanning
52.0XX

 Scan Abutment
on Ti base
53.100 53.200

Name Library		Implant Type	Analogue	Scan Abutments	
Camlog®	DESS - CL Pre-milled CoCr (Desktop or Intraoral)	3.3	14.064	52.064 53.100	
		3.8	14.065	50.065 52.065 53.100	
		4.3	14.066	50.066 52.066 53.100	
		5.0	14.067	52.067 53.100	
Conelog®	DESS - CL COLPre-milled CoCr (Desktop or Intraoral)	NP	14.074	52.074 53.100	
		RP	14.075	52.075 53.100	
		WP	14.076	52.076 53.100	
Biohorizns® External	DESS - BH EXT Pre-milled CoCr (Desktop or Intraoral)	RP/4.2	14.021	50.021 52.021 53.100	
		WP/5.2	14.022	50.022 52.022	
MIS® Seven	DESS - MIS SVN Pre-milled CoCr (Desktop or Intraoral)	SP	14.095	50.017 52.017 53.100	
		WP	14.096	50.018 52.018 53.100	
Megagen AntRidge®	DESS - MG Pre-milled CoCr (Desktop or Intraoral)		14.057	50.057 53.100	
Dentium Superline™ & Implantium	DESS - DT Pre-milled CoCr (Desktop or Intraoral)	RP	14.090	52.090 53.100	

 non-engaging

 non-engaging

 Scan Abutment
for desktop scanning
50.0XX

 Scan Abutment
for intra-oral scanning
52.0XX

 Scan Abutment
on Ti base
53.100 53.200

All DESS products are manufactured according to ISO 9001:2015 and ISO 13485:2016 with CE marking for all products Class IIb

All DESS® AURUMBas® screws and screwdrivers are based on the standard ISO 10664

3i® and 3i Certain®, Miniplant®, MicroMiniplant™ and Osseotite® are registered trademarks of 3i Implant Innovations, USA

ASTRA TECH™, OsseoSpeed™, ANKYLOS® C/X, XiVE®, FRIADENT®, ASTRATECH implant system™ EV and UniAbutment

are registered trademarks of DENTSPLY SIRONA Implants

BioHorizons® is a registered trademark of BioHorizons, INC.

Brånemark System®, NobelReplace® and Replace Select™, NobelActive™, NobelReplace® CC, NobelSpeedy™ and Multi-Unit are

registered trademarks of Nobel Biocare AB, Sweden

ITF®, Straumann® and synOcta® are registered trademarks of Straumann Holding AG, Switzerland

MIS® is a registered trademark of MIS Implant Technologies Ltd.

Zimmer Screw-Verband SwissPlus® are registered trademarks of Zimmer Dental INC, USA

Dyna® is a registered trademark of Dyna Dental Engineering BV

Locator® is a registered trademark of ZEST IP Holdings

Camlog® and Conelog® are registered trademarks of CAMLOG Biotechnologies AG

Megagen® is a registered trademark of MEGAGEN IMPLANT CO. LTD

TORX® is a trademark of ACUMENT INTELLECTUAL PROPERTIES, LLC

Bego Semado® is a registered trademark of BEGO GmbH & Co. KG

Anthogyr® is a registered trademark of Anthogyr SAS

Medentis ICX is a registered trademark of medentis medical GmbH

Osstem® is a registered trademark of OSSTEM IMPLANT CO. LTD

Dentium® is a trademark of Dentium Co. Ltd.

GOBALD® is a trademark of Global D SASU.

Biotech Kontakt is a trademark Biotech Dental SAS.

BTI® is a trademark BTI Biotechnology Institute.

DESS products can be used only by licensed dentists, physicians and labs.

Please refer ever to the instructions of use available on the website

www.dess-usa.com

or supplied with the product.

Some products may not be available in your country. Some of the products are cleared for US and Canada.

Please contact your local distributor for more details.

